

PICKING UP OBJECTS WITH THE REACHER



The SUMO comes with a magnet in the rubber foot. This magnet is designed to pick up objects that weigh up to 50 grams. If you are picking up a heavier object, you will need to use the claw. To do this use the SUMO in reacher mode.

MAINTENANCE

Your SUMO comes with a replacement rubber foot as they can wear out with regular use of the stick. To change the worn foot, hold the rubber foot above the lip and simply twist and pull.

To put on the new rubber foot, align end of the stick with the foot opening and push on firmly.

The SUMO can be cleaned by wiping with warm soapy water. Avoid getting the main button and handle wet.

For other tips on using and maintaining your SUMO please visit:
www.supportingmobility.com